



How to fill in results
Direct Antiglobulin Test (4440)

Getting started 1-2-3



The image shows the LabScala login page. At the top is a blue header with the LabScala logo and name. Below the header, the page title is "Login with username and password". There are two input fields: "Username:" and "Password:". Below the "Password:" field is a blue "Login" button and a link "Forgot your password?".

AUTH.01.1




The image shows the LabScala password change page. At the top is a blue header with the LabScala logo and name. Below the header, the page title is "Password change". There are three input fields: "Old password:", "New password:", and "Repeat new password:". Below the "Repeat new password:" field are two buttons: "Change" and "Back".

OS5.2

1. Login with your username and password.
2. If you use LabScala for the first time: Login using the existing Mainio (participant services) password
3. Change the password
 - Password needs to have 9 characters, one of which needs to be a letter and one a number
 - This will be your new administrator password
 - If you are already familiar with LabScala, use your own login name and password to access your account
4. Start using LabScala

Filling results

- The easiest way to access open schemes is from "My EQA" on the front page
- In the view of the next 3 closing schemes the closing date and the status of the scheme can be seen
 - **Open** = eForm open for result filling
 - **Not sent** = result filling started but results not sent to Labquality
 - **Accepted** = results sent to Labquality

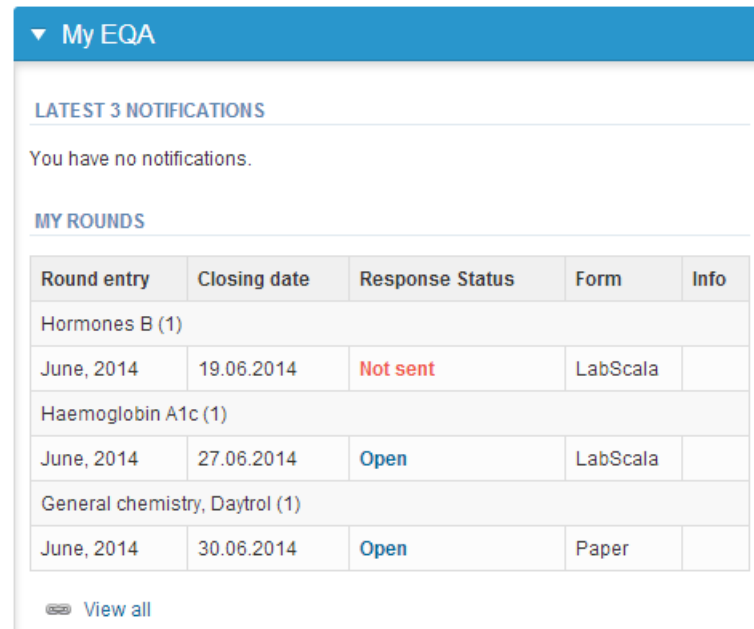

The screenshot shows a user interface for 'My EQA'. It features a blue header with a dropdown arrow and the text 'My EQA'. Below the header, there are two sections: 'LATEST 3 NOTIFICATIONS' and 'MY ROUNDS'. The 'LATEST 3 NOTIFICATIONS' section contains the text 'You have no notifications.'. The 'MY ROUNDS' section contains a table with the following data:

Round entry	Closing date	Response Status	Form	Info
Hormones B (1)				
June, 2014	19.06.2014	Not sent	LabScala	
Haemoglobin A1c (1)				
June, 2014	27.06.2014	Open	LabScala	
General chemistry, Daytrol (1)				
June, 2014	30.06.2014	Open	Paper	

At the bottom of the table, there is a link labeled 'View all' with a small icon to its left.

Filling results

- The eForms can be accessed also from "My Schemes" in the header
 - **Fill results:** access to eForms that are open for result filling
 - **View all:** access to a list of all ordered schemes
 - **Mainio:** other Internet – surveys and e-schemes



Filling results

- After selecting an open scheme the results can be filled
- Result filling is presented as a process:
 - **Request:** Important round-specific information
 - **Instructions:** Printable instruction letter
 - **Pre-analytics:** Sample delivery information
 - **Analytics:** Enter your results
 - **Post-analytics:** Validation of the results (check and save)
 - **Exit:** Back to the front page
- Access to the next step of the process can be done after the previous step is completed

Request

- After opening the scheme the request page is opened with the following information
 - **Product:** here it can be seen what and how much has been ordered
 - **Delivery:** samples sent from Labquality
 - **Opening:** Result filling /eForm is opened
 - **Closing:** eForm is closed
 - **Status:** status of the scheme (Open, Draft, Final as before)
- Also the scheme specific contact information is filled here
 - The contact persons receive reminders and important scheme-specific information
 - **At least one contact person needs to be named!**

Welcome to the round of Antiglobulin test, direct, February, 1-2015 analysis








[? Help](#)

[Request](#)>>[Instructions](#)>>[Pre-analytics](#)>>[Analytics](#)>>[Post-analytics](#)>>[Exit](#)

MY ORDERS

Product	Delivery	Opening	Closing	Status	Form
Antiglobulin test, direct(2)					
	23.02.2015	23.02.2015	12.03.2015	Ordered	LabScala

MY SCHEME SPECIFIC CONTACTS

Name	Send E-mail notification to	
test		
		 
anja	testi@testi.fi	 
anja2	testi@testi.fi	 

► Coordinator contact information


Next

Exit

Filling results

- First add your **scheme-specific** contact info by pressing plus – sign
- Fill in name and email address or phone number (you can add as many contact persons as you like)
- **Save** and choose Next

MY SCHEME SPECIFIC CONTACTS

Name	Send E-mail notification to	
		

[Next](#) [Exit](#)

Add scheme contact person

▼ Add/edit scheme contact person

* Name:

Email:

Phone:

[Back](#)

Instructions

- In Instructions the scheme-specific instructions can be read and printed by pressing the "Print instructions" button on the bottom of the page

LABQUALITY

-
-
-
-

External Quality Assessment Schemes

-
-
-

• **Direct Antiglobulin Test 1, 2015**

-

Welcome to this Direct Antiglobulin Test round.

-

-

Specimens

-

Sample 1 (S001: LQ714115011) and **sample 2** (S002: LQ714115012) are 3-5% red cell suspensions, volume 3 mL.

-

-

Quality control specimens derived from human blood must be handled with the same care as patient samples, i.e. as potential transmitters of serious diseases. The specimens are found to be HBs-Ag and HIV-Ag negative when tested with licensed reagents, but no known test method can offer complete assurance that the specimens will not transmit these or other infectious diseases.

Pre-analytics

- In Pre-analytics, the following is asked
 - **Sample arrival date**
 - **Quantity received:** How many sample sets were received
 - **Sample storage condition:** How have the samples been stored before analysis (refrigerator, room temperature, freezer, other)
 - **Sample preparation date:** if done, if not, can be left empty
- Comments can be saved if needed
- Move forward by selecting "Save & next"




Antiglobulin test, direct, February, 1-2015 pre-analytics

 Help

Request>>Instructions>>**Pre-analytics**>>Analytics>>Post-analytics>>Exit

Sample registration

▼ Sample registration

Product	Has eForm	Code	Quantity ordered	Sample arrival date	Quantity received	Sample storage conditions	Sample preparation date
Antiglobulin test, direct, February, 1-2015	Yes	4440	2	03.02.2015 	2	+2 ... +8 C 	

COMMENTS

Save & next

Exit

Results (1)

In Analytics page the results can be filled

Antiglobulin test, direct, February, 1-2015 - Sample set 1 (Samples)

<<Previous step>>Antiglobulin test, direct>>Next step

Samples Sample sets = number of the samples ordered

SAMPLE SETS

1 2

Sample S001

Sample S002

1. Mark down first the measurement date
2. Choose the method from the method list
3. After method selection the 'reaction strength' box will appear. Select the reaction strength. If you choose 'Other method, please specify the method in the Comment field (in the bottom of the page)
4. Mark down the interpretation of the sample
5. Save the results and the program gets you automatically the sample 2.

Antiglobulin test, direct measurement results for Sample S001

* Measurement date	Antiglobulin	* Method	* Reaction strength
18.02.2015	IgG + C3d	Grifols DG Gel	4+
18.02.2015	IgG	Grifols DG Gel	- Choose
18.02.2015	C3d		
18.02.2015	Result interpretation		Positive

COMMENTS










Results (2)

- There is place for comments at the end of the page
- Options for saving on this page are
 - Use "Save as draft", if you don't want to send the results yet
 - Use "Save as final", if you want to send the results to Labquality
- Back to list –button takes you back to the front page
- If "Save as final" was selected the results are saved and the table is closed. The data can be edited by selecting "Edit data" from the bottom of the page

Sample S001

Sample S002

▼ Antiglobulin test, direct measurement results for Sample S001

* Measurement date	Antiglobulin	* Method	* Reaction strength
18.02.2015 	IgG + C3d	Grifols DG Gel <input type="text"/>  	4+ <input type="text"/>
18.02.2015 	IgG	Grifols DG Gel <input type="text"/>  	- Choose <input type="text"/>
18.02.2015 	C3d	<input type="text"/> 	
18.02.2015 	Result interpretation		Positive <input type="text"/>

COMMENTS

Post-analytics (1)

- When all of the results have been filled the user is taken to Post-analytics for result validation
 - **Product:** what part of the process is being validated
 - **Sample set:** which sample set is being validated
 - **Sample:** which sample
 - **Errors:** if there are analyte-specific validation limit errors these are shown here
 - **Last saved:** The user who has saved the results
 - **Date:** date of last saving
 - **Status:** status of the results (Accepted, Draft, Open, Error)

Antiglobulin test, direct, February, 1-2015 postanalytics

[Help](#)

Request>>Instructions>>Pre-analytics>>Analytics>>Post-analytics>>Exit

Validate results

Validation results

Product	Sample set	Sample	Errors	Last saved	Date	Status
Antiglobulin test, direct	1	Sample S001	OK	Pakkanen, Anja	18.02.2015 13:46	Accepted
Antiglobulin test, direct	1	Sample S002	OK	Pakkanen, Anja	03.02.2015 14:38	Accepted
Antiglobulin test, direct	2	Sample S001	OK	Pakkanen, Anja	03.02.2015 14:39	Accepted
Antiglobulin test, direct	2	Sample S002	OK	Pakkanen, Anja	18.02.2015 13:46	Accepted

Accept and send results

Exit

Post-analytics (2)

- The results can be edited by selecting the correct sample
- When all of the states are such that the results can be sent to Labquality, select "Accept and send results"
- If you wish not to send the results yet, select "Exit"
- The results can be edited as long as the round is open, even if "Accept and send results" has been selected

Antiglobulin test, direct, February, 1-2015 postanalytics

[? Help](#)

[Request](#)>>[Instructions](#)>>[Pre-analytics](#)>>[Analytics](#)>>**[Post-analytics](#)**>>[Exit](#)

Validate results

▼ Validation results

Product	Sample set	Sample	Errors	Last saved	Date	Status
Antiglobulin test, direct	1	Sample S001	OK	Pakkanen, Anja	18.02.2015 13:46	Accepted
Antiglobulin test, direct	1	Sample S002	OK	Pakkanen, Anja	03.02.2015 14:38	Accepted
Antiglobulin test, direct	2	Sample S001	OK	Pakkanen, Anja	03.02.2015 14:39	Accepted
Antiglobulin test, direct	2	Sample S002	OK	Pakkanen, Anja	18.02.2015 13:46	Accepted

Accept and send results

Exit

Exit

- After selecting "Exit" you will be taken back to the front page

LabScala welcomes you !




[? Help](#)

▼ My EQA

LATEST 3 NOTIFICATIONS






You have no notifications.

MY ROUNDS


Round entry	Closing date	Response Status	Form	Info
Parasites in faeces, virtual microscopy (1)				
April, 2014	18.06.2014	Not sent	LabScala	
Rotavirus and adenovirus antigen detection (4)				
June, 2014	19.06.2014	Not sent	LabScala	
Coeliac disease (1)				
June, 2014	26.06.2014	Not sent	LabScala	

▼ Shortcuts & messages

SHORTCUTS

-  [Place orders](#)
-  [View reports](#)
-  [e-schemes](#)
-  [Fill results \(Mainio\)](#)
-  [Go to Mainio](#)

MESSAGES

 **LabScala videos on YouTube**

As more and more functionality is added to our EQA-portal LabScala, the need for userguidance is for sure needed.

In order to help our customers use LabScala we have created short LabScala-videotutorials for the most used functions: login, adding users, adding devices, how to use the eForms etc. The videos can be found on LabScala's own YouTube account [@labscalafinland](#) on the page

LabScala buttons



Save

Enables you to save changes on the form



Back

Takes you back to the previous view



Enables you to add some information. In tables it adds a row.



Edit button enables you to edit texts and information



Delete button enables you to delete texts and information



Accept button marks something as being accepted or valid



Lookup button marks a search field where you can enter text to be searched for




List button marks a field where you can search from the background register



Takes you back to the front page

Questions?

- In case you have questions, please contact:
Anja Pakkanen
anja.pakkanen@labquality.fi
- In order to help our customers use LabScala we have created short LabScala-videotutorials for the most used functions: login, adding users, adding devices, how to use the eForms etc. The videos can be found on LabScala's own YouTube-account at <https://www.youtube.com/user/LabScala>
- You can find these LabScala instructions behind the Help button in the front page of LabScala

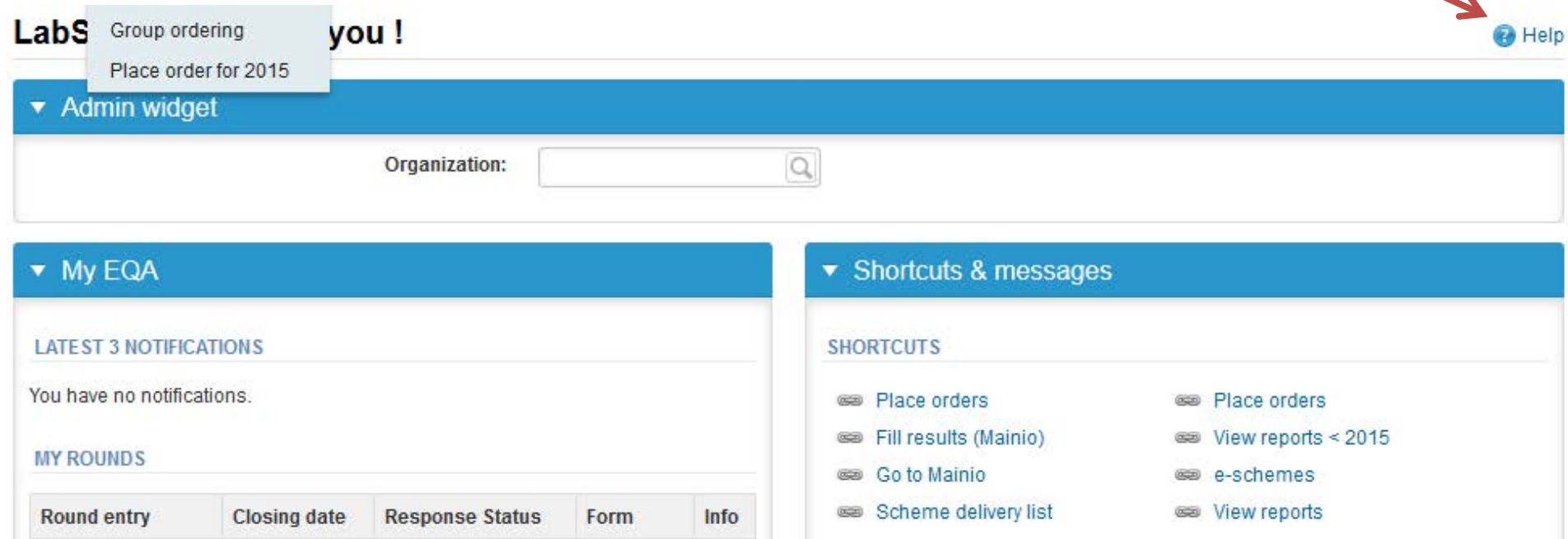

The screenshot shows the LabScala user interface. At the top left, the text "LabS" is partially visible, followed by a notification box containing "Group ordering" and "Place order for 2015". To the right of this is the text "you!". In the top right corner, there is a "Help" button with a question mark icon, which is pointed to by a red arrow from the text in the third bullet point. Below the header, there is a blue bar with a dropdown arrow and the text "Admin widget". Underneath this is a search bar labeled "Organization:" with a magnifying glass icon. The main content area is divided into two columns. The left column has a blue header "My EQA" and contains sections for "LATEST 3 NOTIFICATIONS" (stating "You have no notifications.") and "MY ROUNDS" (with a table below). The right column has a blue header "Shortcuts & messages" and contains a "SHORTCUTS" section with two columns of links, each preceded by a lock icon. The table under "MY ROUNDS" has the following structure:

Round entry	Closing date	Response Status	Form	Info
-------------	--------------	-----------------	------	------

The shortcuts section lists the following items:

- Place orders
- Fill results (Mainio)
- Go to Mainio
- Scheme delivery list
- Place orders
- View reports < 2015
- e-schemes
- View reports